

# AutoMAX GM390

## **Alginate Mixer**

Smooth mixture in seconds for a quick and accurate impression

Consistent bubble-free mixtures

Safety microswitches and anti-vibration mechanisms

Easy to clean and maintain







# AutoMAX GM390 Alginate Mixer

AutoMAX is a high performance alginate mixer with patented technologies that improve the quality and safety of the product, including safety microswitch controls and anti-vibration mechanisms, allowing users to mix alignate powders into a homogeneous paste in just seconds. With this unit, clinics can easily standardize impression procedures and eliminate human error while getting a smooth mixture every time.



### Simple steps to creating a perfect impression:



 Add alginate powder and water into the mixing cup.

2. Moisten with the stirrer for a few seconds and close the mixing cup.



3. Put the mixing cup into the mixing well.

4. Cover the lid and press" START".





After mixing, use the spatula and remove the mixed impression.

### 3 Default Mixing Modes

Scoops	Mixing Time
•	8 secs
-	10 secs
	12 secs

#### Specifications:

• Timer Range 1 ~ 16 seconds

• Mixing Modes 3 (Default: 8, 10, 12 secs)

• Speed 3,400 rpm

Power Input AC 110 or 230V, 50/60 Hz
 Dimensions 250(L) x 205(W) x 290(H) mm

• Net Weight 17.5 kg

• Accessories Spatula/Stirrer/Scoop/Accessory Holder/

Water Dosing Bottle/Mixing Cups(x4)















5F, 70, Sec. 1, Guangfu Rd., Sanchong Dist., New Taipei City 24158, TAIWAN TEL: + 886 2 29993869 FAX: + 886 2 29993647

Email: monitex@ monitex.com.tw

www.monitex.com.tw





